Rx only

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MGK1XXYY

MGK1RXXYY

EN BIOPSY NEEDLE FOR SOFT TISSUE FOR PALIUM GUN AND COMPATIBLE WITH BARD MAGNUM® GUN

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ENGLISH

Indications and Description

1. Before use, make sure that packaging is not opened and/or damaged.
2. Check that the length and diameter of the needle corresponds to what is indicated on the label.
3. Open the packaging and remove the protection cap.
4. At the doctor's discretion, apply a local disinfectant to the needle insertion area and administer a local anesthetic. To facilitate the insertion of the needle, it is possible to make a skin incision using a scalpel.
5. Prepare the gun following the instructions regarding cleaning and sterilization included in the gun packaging.
6. Arm the actuation handle twice to prepare the device for collection (this automatically inserts the safety device).
7. Position the lever relative to the stroke on 15mm or 22mm.
8. Open the lid of the previously loaded gun, insert the needle and close the lid (the lid will remain partially raised due to the spacer still inserted). Keeping the lid closed, remove the needle guard by closing the gun completely.
9. Insert the needle into the patient.
10. Disengage the safety by turning the safety release button located on the side of the device.
11. To activate the device, use the firing trigger. This triggers the spring mechanism, which pushes the needle forward to collect the biopsy sample.
12. After removing the needle from the patient, arm the activation handle once to expose the biopsy opening located at the distal end of the needle. In this way, the sample is exposed, allowing it to be recovered. Note: to recover the sample, there is no need to remove the needle from the device.
13. Check that the gun is loaded, otherwise arm the activation handle a second time, to prepare the device for a further biopsy. Before proceeding, check the integrity of the needle.
14. If you do not need to perform further biopsies, you can remove the needle by opening the lid and reinserting the spacer.
15. If the needle is equipped with NeedleVISE®, place the NeedleVISE® on a stable surface and insert the tip of the needle into the NeedleVISE®.
16. Throw the needle into the needle container according to specific hospital procedures.

Warnings and Precautions:

To be used only by specialized medical practitioners. The procedure requires that the operator is familiar with the techniques of anesthesia according to the method of electric neurostimulation. The using procedure has an informative purpose, specialist physician is responsible for the proper procedure and techniques to be used with this device; eventual variations are at specialist physician's discretion. Verify the expire date before use. Used medical devices have a high contamination risk, be very careful to avoid accidental puncture. Treat the medical devices as contaminated hospital waste. Dispose of it in accordance with applicable laws and regulations. Do not use for application other than soft tissue biopsy.

ATTENTION THE NEEDLE IS NOT SUITABLE FOR USE UNDER MRI (Magnetic Resonance Imaging)

This product has been explicitly designed and manufactured for single use only. The reuse can cause the loss of mechanical properties and invalidate device's functionality. The company M.D.L. will not be held responsible in any way for the consequences in case of reuse of the device

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